



FreeStyle

Freedom Lite

Blood Glucose Monitoring System

1 Wash hands.

2 Hold lancing device.

3 Insert strip.

4 Turn dial.

5 Press button 3x.

6 Insert strip.

7 Turn dial.

8 Lancing device with 4 levels of depth.

9 Press button.

10 Insert strip into meter.

11 Meter with strip inserted.

12 Meter display.

13 Insert strip into lancing device.

14 Lancing device at  $> 45^\circ$ .

15 Meter with strip inserted.

16 Meter display showing 5.8.

17 Dispose of used strips in a red bin.

Abbott Diabetes Care

Klantenservice

0800 - 022 88 28 (gratis)

 **Abbott**  
Diabetes Care